
WM 68240a 12/2016 DE, EN, FR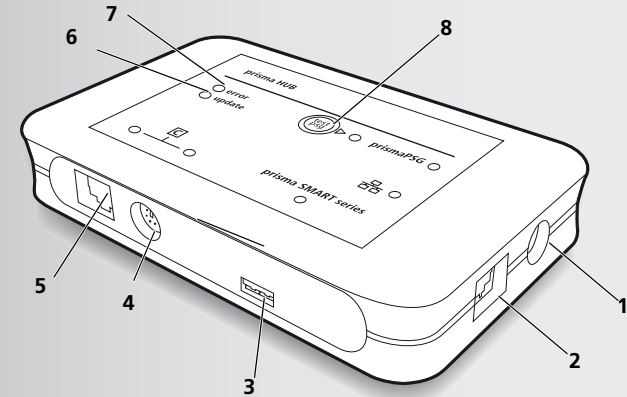

**Löwenstein Medical
Technology GmbH + Co. KG**
Kronsaalsweg 40
22525 Hamburg, Germany
T: +49 40 54702-0
F: +49 40 54702-461
www.loewensteinmedical.de


prisma HUB

Kommunikationsmodul für Therapiegeräte des Typs
WM 090 TD / WM 100 TD / WM 110 TD / WM 120 TD
Communication module for therapy devices of type
WM 090 TD / WM 100 TD / WM 110 TD / WM 120 TD
Module de communication pour les appareils de thérapie
du type WM 090 TD / WM 100 TD / WM 110 TD /
WM 120 TD


1

prismaLINE series (WM 100 TD / WM 110 TD / WM 120 TD)
+ prismaConnect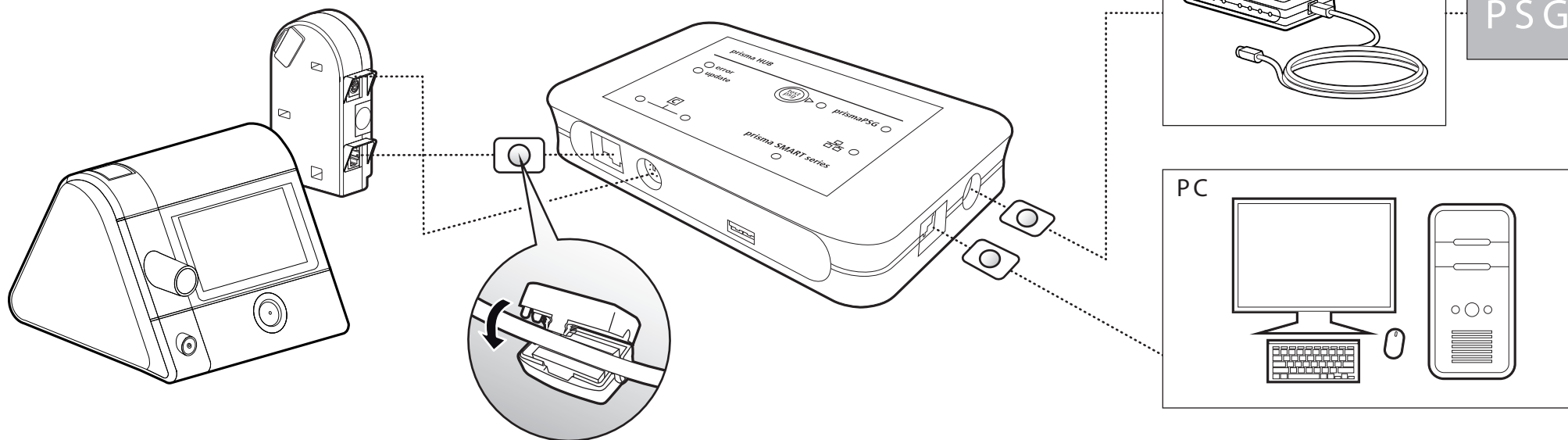

2

prisma SMART series (WM 090 TD)

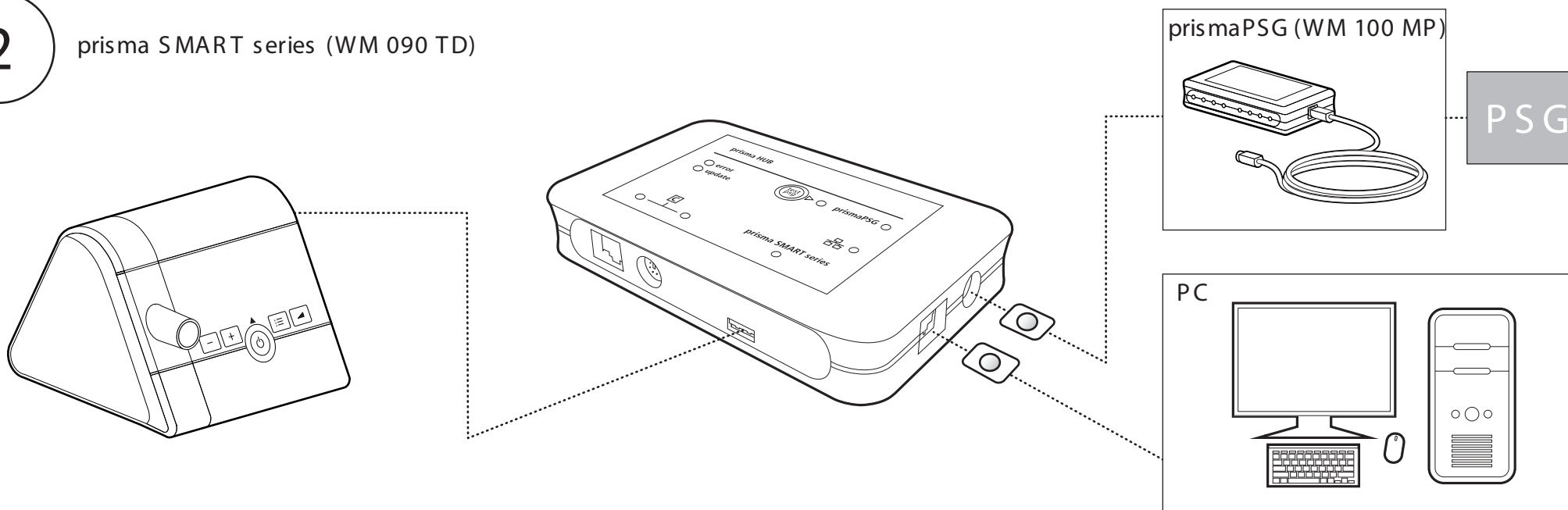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11 Technical data

Product class to 93/42/EEC	I
Dimensions W x H x D in cm	15 x 3 x 9.2
Weight	0.2 kg
Temperature range operation storage	+5 °C to +40 °C -25 °C to +70 °C
Permitted humidity for operation and storage	15 % to 93 %, no condensation
Air pressure range	700 hPa - 1060 hPa, corresponds to an altitude of 3000 m above MSL
Classification to EN 60601-1-11: type of protection against electric shock	Protection class II
Classification to DIN EN 60601-1: operating mode	Continuous duty
Electrical rating	12 V/24 V
Power capacity	2 VA
Insulation of Ethernet port to PC	4 kV
Insulation of Ethernet port to prismaCONNECT module	1.5 kV

The right to make design modifications is reserved.

12 Warranty

Löwenstein Medical gives the customer a limited manufacturer warranty on a new genuine Löwenstein Medical product and on any replacement part fitted by Löwenstein Medical in accordance with the warranty conditions applicable to the product in question and in accordance with the warranty periods from date of purchase listed below. The warranty conditions are available on the website of the manufacturer. We will also send you the warranty conditions on request. In the event of a claim under warranty, contact your specialist dealer.

Product	Warranty periods
Devices including accessories (except masks)	2 years
Masks including accessories, rechargeable batteries, batteries (unless quoted differently in the technical documentation), sensors, patient circuits	6 months
Disposable products	None

13 Declaration of conformity

Löwenstein Medical Technology GmbH + Co. KG, Kronsaaßweg 40, 22525 Hamburg, Germany, the manufacturer of the devices described in these Instructions for Use, hereby declares that the product complies with the respective regulations of Medical Devices Directive 93/42/EEC. The unabridged text of the Declaration of Conformity can be found on the manufacturer's website.

1 Introduction

1.1 Intended use

The WM 090 MC module is used to create a link between the WM 090 TD therapy device and a PC via Ethernet. It is also used to create a link between the WM 090 TD therapy device and the prismaPSG module of type WM 100 MP. The WM 090 MP module is designed solely for use indoors. The WM 090 MP module is not designed for domestic use.

2 Safety

2.1 Safety information

Risk of injury from device or component malfunctions!

A damaged device or damaged components may injure the patient, the user and people in the vicinity.

⇒ Only operate the device and its components if they are externally undamaged.

⇒ Only operate the device and its components if a function check has been completed successfully.

Risk of injury from operating the device outside the specified ambient conditions!

Using the device outside the specified ambient conditions may lead to tolerances being exceeded and to device failure and injury to the patient.

⇒ Only operate device within the specified ambient conditions (see section entitled "Technical data").

⇒ Only operate the device in closed rooms.

2.2 General information

- Connect only the devices and modules permitted in accordance with these Instructions for Use. Devices connected to the LAN port of the module must meet their respective product standard. Non-medical equipment should be positioned out of the patient's vicinity.
- The owner/operator is responsible for ensuring the compatibility of the therapy device and of all the components or accessories connected to the patient before use. Have modifications to the device carried out only by the manufacturer, Weinmann, or by specialist staff expressly authorized by the manufacturer.
- If the WM 090 MC module is integrated in an IT network which includes other devices, the owner/operator must check whether this causes risks to patients, operators or third parties (IEC 80001-1).

- The use of third-party articles may lead to incompatibility with the device. In such cases, please be aware that any claim under warranty and liability will be void if neither the accessories nor the genuine replacement parts recommended in the Instructions for Use are used.
- To prevent infection or bacterial contamination, follow the section about hygiene treatment (see section entitled "Hygiene treatment").

3 Product description

3.1 Overview

A diagram of the individual parts can be found on the title page.

Key

- Connection for prismaPSG (WM 100 MP)
- LAN port for connecting to a PC.
- Port for prisma SMART series (WM 090 TD)
- PSG port to the prismaCONNECT module (WM 100 MC).
- LAN port to the prismaCONNECT module (WM 100 MC).
- Update LED - indicates an update in progress.
- Error LED - indicates a fault.
- PSG key - outputs test signals to the prismaPSG.

3.2 Marks and symbols

Position	Symbol	Description
2		LAN port to the PC
4, 5		Port to the prismaCONNECT module

3.3 Symbols on the device ID plate

	Do not dispose of device in domestic waste.
	Degree of protection against electric shock: protection class II device
	Protect from wet
	Date of manufacture and manufacturer's address
Type:	Type designation: device of WM 090 MC type
	Follow Instructions for Use.
SN	Serial number
CE	CE marking

4 Operation

4.1 Connecting the module

- Make plug connections in line with **Figure 1** or **Figure 2**.

You can use the module either with a therapy device of the prisma SMART series or with a therapy device of the prismaLINE series (in conjunction with the prismaCONNECT module). Do not connect two therapy devices simultaneously.

The relevant LED comes on.

- Attach ferrites to the cables in line with **Figure 1** or **Figure 2**, attaching the ferrites as close as possible to the prisma HUB module.

4.2 Switching on the module

- To switch on the module: connect the module to a therapy device.
The LEDs which have a plug connection come on. The LEDs go out one minute after the last change.
- To switch on the LEDs: press PSG key **8**.

4.3 Outputting a PSG test signal

- Press PSG key **8**.
The LED comes on. A synthetic signal is output to the PSG system for 2 minutes.

4.4 Configuring the module

The DHCP preset is activated at the LAN port for connection to a PC **2**. A dynamic IP address is assigned to the prisma HUB module. If no DHCP server is available, the IP address is assigned by zeroconf/auto-IP in the range 169.254.1.0 to 169.254.254.255.

You can assign a static IP address to the module with the aid of the prismaHUBconfig PC software.

4.5 Integrating the module in an IT network

The module can be integrated in an IT network to create a link between the module and a PC. The network must have the following properties:

- UDP and TCP ports 51337 and 51338 must be open and may not be blocked by a firewall.
- The UDP protocol is required to identify devices in the network automatically, data exchange subsequently takes place using the TCP protocol.

If the IT network fails, you can continue operating the therapy device via the display and transmitting data via SD card. This prevents failure of the IT network giving rise to hazardous situations.

5 Hygiene treatment

5.1 General information

- Wear appropriate safety gear for the disinfecting process.
- Refer to the Instructions for Use for the disinfectant used.

5.2 Intervals

Clean the module weekly. If required (e.g. following infectious diseases or severe contamination), disinfect the module. Sterilizing is not permitted.

5.3 Subjecting module to a hygiene treatment

NOTICE

Risk of material damage as a result of ingress of liquids!

The module may be damaged by the ingress of liquids.
⇒ Do not immerse the module in liquids.

- Detach all connections from the module.
- Wipe the module over with a soft cloth.

- If necessary: remove more stubborn particles of dirt with a soft cloth moistened with water or disinfectant (recommended products: terralin® protect or perform advanced Alcohol EP).

6 Function check

Carry out a function check after every hygiene treatment and repair, but at least every 6 months.

- Remove all cables from the module one after another.
The relevant LED goes off.
- Re-connect all the cables to the module one after another.
The relevant LED comes on.
- Press **PSG** key **8**.
The LED comes on. A synthetic signal is output to the PSG system for 2 minutes.
- If the module is not working properly or is damaged: contact your specialist dealer.

7 Troubleshooting

Fault	Cause	Remedy
Error LED 7 is on	Technical fault	Have module repaired.
No LAN connection possible	Cable plugged in incorrectly	Check whether the LAN cable is in the socket marked
No connection to the prisma CONNECT module	Cable plugged in incorrectly or not at all.	Check whether the PSG cable and the LAN cable are in the sockets marked and the LEDs are on.
No connection to the prisma-PSG module	Cable plugged in incorrectly	Check whether the PSG cable is in the socket marked prismaPSG and the LED is on.
No therapy device is found in the prismaTS software	IP address of the prismaHUB configured incorrectly	Check IP settings with the aid of the prismaTS software.
	The prismaCONNECT module was configured to a static IP address	Configure prisma CONNECT module to DHCP
	Firewall blocking data transmission	Enable UDP and TCP ports 51337 and 51338.

8 Servicing

The module is designed for a service life of 6 years. If used in accordance with its intended use, the module requires no maintenance according to these Instructions for Use.

9 Scope of supply

Part	Item number
prisma HUB, basic device	WM 31665
HUB connecting cable	WM 31664
PSG module connecting cable	WM 29696
3 m CAT-6 patch cable, green	WM 98273
Ferrite (3x)	WM 31669
Key for ferrite	WM 31659
Instructions for Use	WM 68240

10 Accessories and replacement parts

A current list of accessories and replacement parts can be ordered on the internet site of the manufacturer or through your authorized specialist dealer.

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