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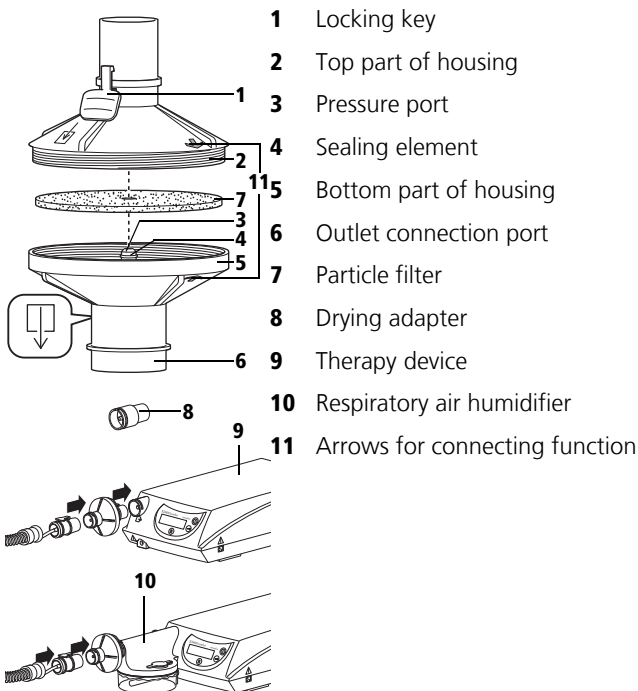
Legend

1.1 Intended purpose

The bacteria filter serves to filter respiratory air on therapy devices for sleep-related respiratory disorders and ventilatory insufficiency to prevent devices becoming contaminated. The maximum duration of use for the particle filter is 24 h.

1.2 Description of function

The bacteria filter is located between the device and the tube system. This includes a particle filter which filters the patient's respiratory air. The filtering action of the particle filter is achieved by means of electrostatic charging.

2. Safety information

2.1 Safety regulations

For your own safety, the safety of your patients, and to comply with the requirements of EU Directive 93/42/EEC, please observe the following points:

- Please read these instructions for use carefully. They are a constituent part of the device and must be available at all times.
- Use the bacteria filter only for the intended purpose described (see "1.1 Intended purpose" on page 20).
- For each new patient, a new particle filter should always be used and the housing disinfected/sterilized both inside and out. Please

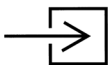
see section entitled “5. Hygienic preparation” on page 25 to avoid an infection or bacterial contamination.

- If third-party items are used, functional failures may occur and fitness for use may be restricted. Biocompatibility requirements may also not be met. Please note that in such cases, any claim under warranty and liability will be voided if neither the accessories nor genuine replacement parts recommended in the instructions for use are used.


2.2 Special marks on the device

Connecting position

Device inlet

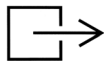Device outlet

Particle filter only

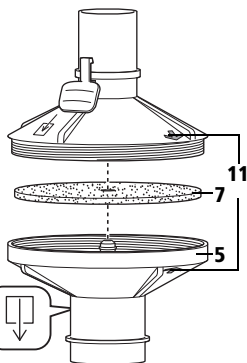
Disposable item

3. Setting up and assembling the device

3.1 Fitting the bacteria filter

A particle filter **7** must be inserted before use. Please proceed as outlined below:

1. Unscrew the two halves of the housing by rotating to the left.

2. If necessary, remove old particle filter **7** from bottom part of the housing **5**.

3. Insert new particle filter **7**. Ensure that particle filter **7** lies evenly on the ribs.

4. Push the two halves of the housing together so that the arrows for connecting position **11** are opposite one another. Then screw the halves of the housing up to the stop. In the final position, the webs for the handles of the two halves of the housing are opposite one another.

3.2 Combining with other devices

You must follow the instructions for use for the combined devices.

Therapy devices

The bacteria filter can be combined with the following therapy devices: SOMNOcomfort, SOMNOsoft +, SOMNOvent S, SOMNOvent ST, SOMNOsmart 2, SOMNOset, SOMNOvent CR, SOMNOvent auto-S, SOMNOvent auto-ST, BiLevel ST 22, VENTIllogic, VENTIllogic LS, VENTIllogic plus.

Tube system

The bacteria filter can be combined with the following tube systems:

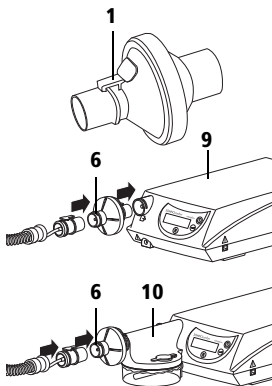
tube system (can be disinfected) WM 24130

tube system (can be sterilized) WM 24120

Respiratory air humidifier

The bacteria filter can be combined with the SOMNOclick and SOMNOclick 300 respiratory air humidifiers.

3.3 Connecting the bacteria filter

1. If necessary, disconnect the corrugated tube from the therapy device by pressing the locking key of the tube adapter.
2. Place the bacteria filter on the therapy device **9** outlet or on the humidifier **10**. Ensure that locking device **1** engages.
3. Click the adapter of the corrugated tube into outlet connection port **6** of the bacteria filter.

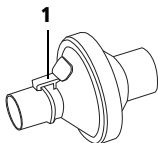
4. Operation

4.1 Starting up

1. Start up the device combination in accordance with the instructions for use for the therapy device.

4.2 After use

1. Switch off the therapy device.
2. Take off the corrugated tube by pressing the locking key.
3. Take off the bacteria filter by pressing locking key **1**.
4. Clean the bacteria filter as described under "5. Hygienic preparation" on page 25.

Note!

To avoid contamination, always use a new particle filter after:

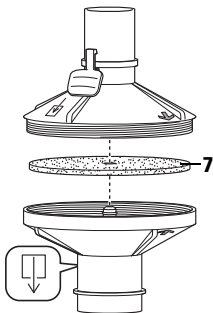
- max. 24 h use
- opening the bacteria filter for cleaning purposes

5. Hygienic preparation

Note!


This product may contain disposable items. Disposable items are intended to be used only once. So use these items only once and do **not** reprocess them. Reprocessing disposable items may impair the functionality and safety of the product and lead to unforeseeable reactions as a result of ageing, embrittlement, wear, thermal load, the effects of chemical processes, etc.

5.1 Critical times



- Particle filter **7** must be changed after 24 h (3 nights of 8 h each).
- The housing of the bacteria filter should be cleaned after 24 h (3 nights of 8 h each).
- Particle filter **7** must always be changed and the housing disinfected or sterilized whenever there is a new patient.
- The pressure measuring tube of the tube system should be changed every four weeks on hygiene grounds.

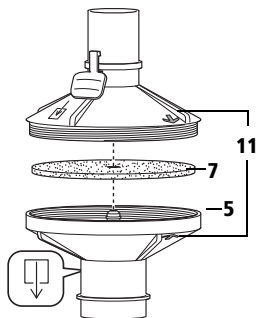
5.2 Cleaning

1. Unscrew the two halves of the housing by rotating them to the left.
2. Remove particle filter **7**.
3. Clean the parts of the housing in a dishwasher at 65 °C, using the top basket of a dishwasher and positioning the housing so that all the areas of the top and bottom part can be thoroughly washed.
4. Rinse the parts of the housing afterwards in clean water.
5. Shake the parts of the housing vigorously to remove water.
6. Push the two halves of the housing together so that the arrows for connecting position **11** are opposite one another. Then screw the halves of the housing up to the stop. In the final position, the webs for the handles of the two halves of the housing are opposite one another.

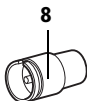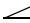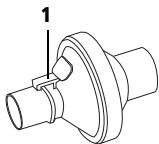7. Plug the red drying adapter **8** (included in the scope of supply of the therapy device) onto the therapy device.
8. Plug the bacteria filter onto drying adapter **8**.
9. Activate drying mode by pressing the Softstart  and On/Off  keys simultaneously (see "Cleaning and disinfecting" in the instructions for use for the therapy device). The device now switches on and dries the pressure measuring section. The time remaining is displayed. The device switches off automatically after 30 minutes.

10. Press locking key **1** and take the bacteria filter off drying adapter **8**.
11. Take drying adapter **8** off your therapy device by twisting slightly.
12. After cleaning, perform a function check (see "6. Function check" on page 28).

13. We recommend changing the housing parts every 3 months.

5.3 Disinfecting

You can disinfect the top and bottom parts of the bacteria filter. Follow the instructions for use for the disinfectant you are using.

We recommend GIGASEPT FF. You are recommended to wear suitable gloves (e.g. household or disposable gloves) during disinfection procedures.

- Procedure is as described in the section entitled "5.2 Cleaning" on page 26.
- In deviation from this procedure, the bacteria filter must be thoroughly rinsed with distilled water and then dried after being disinfected.

5.4 Sterilizing

The housing parts can be autoclaved up to 30x at 134 °C in devices to EN 285.

Minimum retention time: 3 min.

Maximum retention time: 18 min.

- Procedure is as described in the section entitled "5.2 Cleaning" on page 26.

- In deviation from this procedure, the bacteria filter should not be rinsed with clean water following sterilization.

5.5 New patients

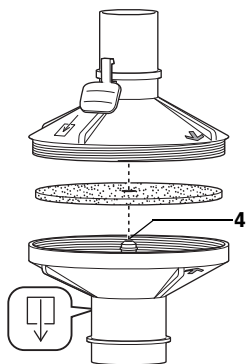
1. The particle filter must always be changed before each new patient.
2. To do so, proceed according to the section entitled “4.2 After use” on page 24.

6. Function check

6.1 Critical times

Perform a function check at least once a month. If you find faults during the function check you may not use the bacteria filter again until the faults have been eliminated.

6.2 Execution

1. Subject the plastic housing and sealing element **4** to a visual inspection: if the bacteria filter is torn or otherwise damaged, it must be replaced completely.

7. Faults and how to eliminate them

Fault	Cause of fault	Eliminating fault
Bacteria filter making loud noises	Housing not correctly screwed together	Screw the bacteria filter in again as described in the section entitled "3.1 Fitting the bacteria filter"
	Cracks in bacteria filter housing	Replace the bacteria filter
No pressure or incorrect pressure building up at the mask	Water in pressure measuring duct	Dry pressure measuring duct ("5. Hygienic preparation")
	No mains voltage at the therapy device	Check that mains connection cable is securely attached. If appropriate, check function of socket by connecting another device (e.g. lamp) to it.

8. Maintenance

The bacteria filter requires no maintenance.

If you find faulty parts during the function check, please replace them.

8.1 Disposal

If the bacteria filter can no longer be used, please dispose of it via your hospital's waste disposal service or via domestic waste.

9. Scope of supply

9.1 Standard scope of supply

Bacteria filter, complete	WM 24148
consisting of:	
bacteria filter, assembled	WM 24639
kit, 10 particle filters	WM 15584
instructions for use DE, EN, FR	WM 16790

9.2 Spare parts

Kit I particle filters (10 x WM 24147)	WM 15584
Kit II particle filters (100 x WM 24147)	WM 15576
Kit III particle filters (500 x WM 24147)	WM 15577

10. Technical data

Product class to 93/42/EEC	IIa
Dimensions Ø x L in cm	7.4 x 9.8
Weight	approx. 51 g
Temperature range	
- operation	+5 °C to +40 °C
- storage	- 20 °C to +70 °C
Permitted humidity for operation	≤ 95 % r.h. and storage (no condensation)
Ambient pressure range	700 to 1060 mbar
Increase in noise level 1 m from the device in the patient's position as per EN ISO 17510 max.	0,5 dB(A)
V_T	≤ 1,5l
Max. permitted flow (flowing off freely)	300 l/min
Internal volume of the bacteria filter	85 ml
Therapy pressure range	3 to 35 mbar
Filtration efficacy checked as per DIN EN 13328-1	
Degree of separation / penetration value to ISO 23328-1:	98,1% / 1,9%
Usage time particle filter	≤ 24 h
Material for housing	PC
Housing can be autoclaved in devices to EN 285	134 °C
We reserve the right to make design modifications	

11. Warranty

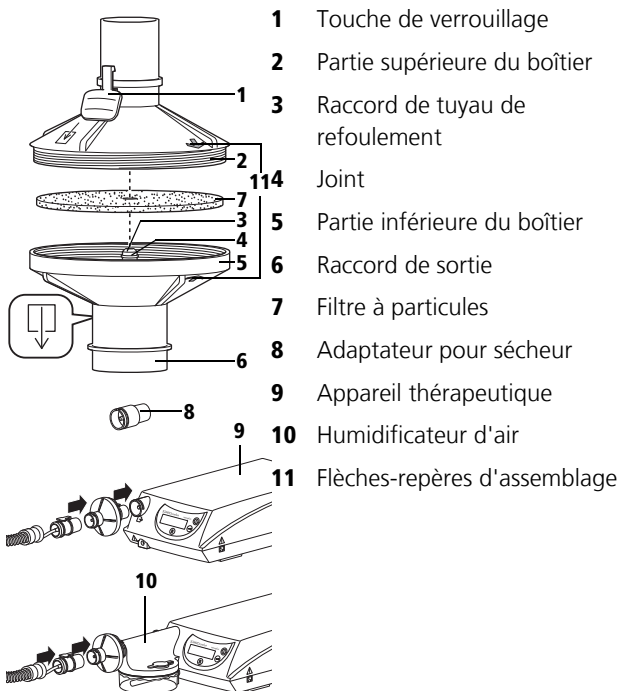
- Löwenstein Medical provides a warranty of 3 months from the date of purchase that the product is free of defects if used in accordance with purpose.
- A condition for claims under warranty is submission of a proof of purchase showing the sales source and date of purchase.
- We will not provide a warranty in the event of:
 - the instructions for use not being followed
 - operating faults
 - incorrect use or treatment
 - failure to use original spare parts
 - third-party intervention in the device for repair purposes by non-authorized persons
 - force majeure
 - transport damage as a result of incorrect packing when returning devices
 - ordinary wear and tear.
- Löwenstein Medical will not be held liable for consequential damage unless due to deliberate fault or gross negligence or in the case of mildly negligent injury to life or limb.
- Löwenstein Medical reserves the right to eliminate the fault, to supply a non-defective item or to reduce the purchase price
- appropriately at its discretion.
- If a claim under warranty is rejected, we will not bear the costs of transport to and from ourselves.
- Statutory claims under warranty remain unaffected.

12. Declaration of conformity

Löwenstein Medical Technology GmbH + Co. KG declares herewith that the product complies fully with the respective regulations of the Medical Device Directive 93/42/EEC. The unabridged text of the Declaration of Conformity can be found on our website at www.loewensteinmedical.de

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